Your Liner[®]

Instructions for Use - *for patients*
Instructions d'utilisation - *pour patients*
Istruzioni per l'uso - *per i pazienti*
Gebrauchsanweisung - *für Patienten*
Gebruiksaanwijzing - *voor patiënten*
Instrucciones para el uso - *para pacientes*
Käyttöohjeet - *potilaille*
Bruksanvisning - *for pasienter*
Brugsanvisning - *til patienter*
Bruksanvisning - *för patienter*
Instrukcja użytkowania - *dla pacjentów*

Your Liner®

ENGLISH	3
DEUTSCH	5
FRANÇAIS	8
ITALIANO	11
NEDERLAND	13
ESPAÑOL	16
SUOMI	18
NORSK	21
DANSK	23
SVENSKA	26
POLSKI	28

<i>Aut.</i>	2023.08.11	CGA	
<i>Rev.</i>	2023.08.11	KMA	
<i>Appr.</i>	2023.08.11	CGA	

Rev6 - 2023.08.11

ID: 4.3.1.2.2_JFU_Patients_SMT-L

Introduction	3
General Disclaimer	3
Information - Intended use	3
1. Donning	3
2. Suspension	3
3. Doffing	4
4. Liner cleaning and care	4
5. Skin care	4
6. Warranty	4
7. Warnings & Precautions	4

Introduction

Dear user, congratulations on starting to use Your® Liner !

Your® Liner was passionately crafted by your MotionTech prosthetic design & manufacturing experts. We exchanged with your prosthetic technician, taking into account the feedback you gave him, to come up with the best design for your specific needs.

We hope you will enjoy wearing it as much as we enjoyed crafting it specifically for you !

Remember that in order to enjoy the optimal performance and comfort while using Your® Liner, it is important to apply the following instructions. Please take the time to review them carefully to make sure you'll make the best of Your® Liner !

General Disclaimer

MotionTech retains all intellectual property rights reflected in its physical products, regardless of the transfer of the physical products to another party or parties.

For safety reasons, Your® Liner can only be ordered by a certified prosthetic technician.

Information - Intended use

Intended use of Your® Liners

A Your® Liner is a protective cover designed to be worn over a residual limb. A Your® Liner is intended to be in direct contact with the intact skin of the stump it was designed for. A Your® Liner is intended to provide a slight compression onto its user's residual limb.

A Your® Liner is intended to be used in conjunction with a prosthetic socket. Its function is to provide cover for the residual limb, and mitigate the load transferred from the residual limb to the prosthesis over the various tissues.

A Your® Liner is intended to be worn for a maximum of 8h in a row. It should then be doffed and cleaned according to cleaning instructions provided in the present booklet.

Intended use of Your® Tibial Cushion Liners

On top of the aforementioned intended use applicable to all Your® Liners, Your® Tibial Cushion Liners are intended only for patients with a transtibial amputation.

A Your® Tibial Cushion Liner does not play a role in the suspension of the prosthesis. It is therefore intended to be used with an adapted suspension system independent from the liner (for example a suspension sleeve).

Intended use of Your® Tibial Suction Liners

On top of the aforementioned intended use applicable to all Your® Liners, Your® Tibial Suction Liners are intended only for patients with a transtibial amputation.

A Your® Tibial Suction Liner is intended to be used with a dedicated prosthetic socket equipped with a standard expulsion valve. A Your® Tibial Suction Liners must also be used with a supplementary suspension system independent from the liner (for example a suspension sleeve), in case the patient presents a short residual limb, substantial daily volume fluctuations, or if sufficient suspension can not be achieved without such supplementary suspension system.

Intended use of Your® Femoral Suction Liners

On top of the aforementioned intended use applicable to all Your® Liners, Your® Femoral Suction Liners are intended only for patients with a transfemoral amputation.

A Your® Femoral Suction Liner is intended to be used with a dedicated prosthetic socket equipped with a standard expulsion valve. A Your® Femoral Suction Liners must also be used with a supplementary suspension system independent from the liner (for example a Silesian belt or a Pelvic band), in case the patient presents a short residual limb, substantial daily volume fluctuations, or if sufficient suspension can not be achieved without such supplementary suspension system.

Intended use of Your® Tibial Pin-Lock Liners

On top of the aforementioned intended use applicable to all Your® Liners, Your® Tibial Pin-Lock Liners are intended only for patients with a transtibial amputation.

A Your® Tibial Pin-Lock Liner is intended to be used with a plunger pin and a locking mechanism. A locking mechanism therefore needs to be integrated in the prosthesis ; and a plunger pin compatible with the chosen locking

mechanism needs to be fastened to the distal umbrella of the Your® Tibial Pin-Lock Liner by the prosthetic technician, prior to provision of the Your® Tibial Pin-Lock Liner to its intended patient.

Intended use of Your® Femoral Pin-Lock Liners

On top of the aforementioned intended use applicable to Your® Liners, Your® Femoral Pin-Lock Liners are intended only for patients with a transfemoral amputation.

A Your® Femoral Pin-Lock Liner is intended to be used with a plunger pin and a locking mechanism. A locking mechanism therefore needs to be integrated in the prosthesis ; and a plunger pin compatible with the chosen locking mechanism needs to be fastened to the distal umbrella of the Your® Femoral Pin-Lock Liner by the prosthetic technician, prior to provision of the Your® Femoral Pin-Lock Liner to its intended patient.

1. Donning

Your® Liner was designed specifically for your limb. It presents a unique geometry and is intended to be carefully positioned and aligned when used in order to offer the comfort that we want you to enjoy.

It is therefore **crucial** that you take the time to position it correctly when donning Your® Liner.

When donning Your® liner, turn it inside out so that the fabric cover is on the inside. Your limb should only be in contact with the bare silicone surface, never with the fabric-covered one.

Grab the liner close to the distal end ; locate the alignment line on the inner surface, and align it carefully so that it will be on the front part (anterior) of your limb when the liner is donned on.

NOTES

- Before donning, check that the inside of the liner is clean, dry and that there are no external objects, substances or items that can cause skin irritation.
- Before donning, check that the liner is not hot, to avoid burning your skin while donning.
- Never don the liner with the fabric cover in direct contact with your skin as this could lead to skin irritations.

Position the distal end of the liner against your limb, and carefully roll it upward onto your leg, paying attention not to leave empty air pockets between the distal end of your limb and the liner. Be careful not to damage the liner with your fingernails. Do not pull the liner up from the proximal end ; as it may stretch your skin and cause discomfort, blisters or rashes.

Once donned on, make sure that there are no wrinkles or air pockets between the liner and your limb. These could cause irritation of your limb.

In order to make sure that you aligned the liner correctly, check that the thicker parts of your liner are correctly covering the anatomical structures they are supposed to, as your prosthetic technician indicated when he provided you with the liner. The alignment line on the inner surface should be positioned towards the front (anterior) of your limb. If this is not the case, start over.

NOTE

The first time you put on your liner, your prosthetic technician should be present to check that it is conform to his design indications. He should also explain which zones are covered by extra thickness for more cushioning and why, in order to help you align your liner when he is not present.

2. Suspension

2.1. Your® Tibial Cushion Liners

A Your® Tibial Cushion Liner is designed to work with a suspension sleeve. Even if your prosthetic technician provided you with a socket that also adds some anatomical suspension, DO NOT use your liner with the rest of your prosthesis without the suspension sleeve. This may result in slippage of the prosthesis on the liner.

2.2. Your® Tibial/Femoral Pin-Lock Liners

Your® Pin-Lock Liners are based on a pin-lock suspension mechanism. Consult your prosthetic technician for additional instructions on how to properly use this type of liners.

Every effort has been implemented to ensure that the suspension strength of Your® Pin-Lock Liners is as high as possible. However, as Your® Pin-Lock Liners are tailor-made, classical strength tests are not sufficient to ensure a flawless mechanical strength for each designed product. If any sign of excessive movement or product wear is detected on or around the locking part of the liner at any time, stop using the device immediately and consult your physician or prosthetic technician.

Do not use Your® Femoral Pin-Lock Liner without a supplementary suspension method if you present a short residual limb. This condition may lead to loss of suspension.

2.3. Your® Tibial/Femoral Suction Liners

Your® Suction Liners are based on a vacuum suspension system. Consult your prosthetic technician for additional instructions on how to properly use this type of liners.

Extensive efforts have been implemented to ensure that the suspension strength of Your® Suction Liners is as high as possible. However, as Your® Suction Liners are tailor-made, classical strength tests are not sufficient to ensure a flawless mechanical strength for each designed product. If any sign of excessive movement or product wear is detected on or around the silicone rings of the liner at any time, stop using the device immediately consult your physician or prosthetic technician.

Do not use a Your® Suction Liner without supplementary suspension method if you present a short residual limb or substantial residual limb volume fluctuations during the day. These two conditions may lead to loss of suspension.

3. Doffing

When doffing Your® Liner, roll it off from the proximal end. Make sure you do not stretch, pinch or pull on the liner to avoid causing any damage to the liner.

NOTE

Do not try to pull your liner by its distal end, as this will create vacuum between your liner and your limb, which may induce various skin problems.

4. Liner cleaning and care

Adequate hygiene is essential to limit risks of irritation, and other skin problems.

4.1. Daily liner cleaning

Your® Liner should be cleaned thoroughly at the end of each day where you wore it.

Use lukewarm tap water and pH neutral soap that does not irritate the skin, and make sure you meticulously wash the interior surface, that comes directly in contact with your skin. Apply the soap with a clean soft cloth or sponge.

Make sure that you rinse all the soap residue from the liner with water, then dry the liner with a clean lint-free cloth.

Place the liner in its normal position, with the fabric cover side out, on a drying stand. Do not let Your® Liner dry inside out.

NOTES

- Do not scrub Your® Liner, as it may roughen the surface of the liner, and cause various skin problems.
- Fabric softeners, bleaches, and other products or cleaning solutions may damage the liner and should not be used.
- Do not clean Your® Liner with products that can cause skin reactions, as residues may remain and harm your skin when using your liner.
- Do not try to dry Your® Liner by using a microwave, oven, dryer, or any electric or heating apparatus, as it may cause damage to Your® Liner. You might also burn yourself when donning your heated liner. Dry Your® Liner only with a clean lint free cloth.

4.2. Weekly disinfection

Place a small amount of ethyl or isopropyl alcohol on a soft, clean lint-free cloth. Softly rub the inside of the liner with the cloth for two minutes. Thoroughly rinse off any excess alcohol with water.

Place the liner in its normal position, with the fabric cover side out, on a drying stand. Do not let Your® Liner dry inside out.

4.3. Daily inspection of your liner

After each day where you used your liner, you should inspect it for indications of damage or wear. These may be signs of changes in the fit of your prosthetic socket. Any damage may reduce the efficiency of the liner and result in potential loss of suspension.

NOTES

- If you use a Your® Pin-lock Liner, you should be especially cautious to observe if damage is present on or around the locking part of the liner. In case of doubt, stop using Your® Liner immediately and report the defects to your prosthetic technician. Failure of the locking mechanism may result in loss of suspension.
- If you use a Your® Suction Liner, you should be especially cautious to observe if damage is present on or around the silicone rings of the liner, and report it immediately to your prosthetic technician. Failure of the silicone rings may result in loss of suspension.

In order to preserve the warranty and ensure your safety while using your prosthesis, contact your prosthetic technician immediately if you notice damage or wear on your liner. The same shall be done if you notice the seam starts unraveling or if the fabric starts to come off the elastomeric body of the liner.

4.4. Storage

If you are not wearing the liner, store it with the fabric cover on the outside, in a cool, dry place. If you do not use it for an extended amount of time, make sure the liner is dry, and cover it with a clean plastic bag.

Do not store Your® Liner under direct sun exposure, as heating and cooling cycles may deteriorate the mechanical performance of your liner.

NOTE

Take care not to expose Your® Liner to glass or carbon fibres or other foreign particles. Such substances can become embedded in the silicone causing skin problems. Washing alone may not be sufficient to eliminate the problem. Return your liner to your prosthetic technician if it is inadvertently exposed to any foreign substances or chemicals.

5. Skin care

The daily use of a prosthesis inherently exposes the skin of your limb to various mechanical stresses. Daily cleaning of your limb is therefore essential. We recommend the use of a mild liquid, pH balanced, 100% fragrance and dye-free soap.

If you notice dry skin, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do not apply lotion to your limb immediately before donning the liner.

NOTE

Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation. Be cautious which products you use !

6. Warranty

6.1. Disclaimer

Swiss Motion Technologies SA warrants that each delivered product will, at the time of delivery, be of workmanlike quality and substantially free of defects. Swiss Motion Technologies SA makes no other warranty, implied or expressed, and makes no warranty of merchantability or fitness for a particular purpose.

This warranty shall terminate immediately upon any action to combine our products with other materials, or in any manner that change the nature of our products. If a replacement product is provided to a customer, the warranty period will be the number of days remaining on the original warranty.

Swiss Motion Technologies SA's liability shall not exceed the purchase price of the product. Swiss Motion Technologies SA is not responsible for any indirect, incidental or consequential damage.

6.2. Warranty conditions

Swiss Motion Technologies SA offers a warranty on each Your® Liner from the date of the seller's invoice for a period of 6 months against defects in material and workmanship. This warranty shall be in lieu of all others, whether expressed or implied by law.

Your® Liners are manufactured based on specifications provided by a prosthetic technician. If a buyer is not satisfied with the product, it may be returned within 30 days of invoice for alteration, replacement, or credit at the sole discretion of Swiss Motion Technologies SA. Credit will only be applied if it is determined the product is not appropriate for the intended patient. Returns will not be accepted for customer errors.

There is no warranty, expressed or implied, for damage caused by abuse, mishandling or accident. The warranty is void if the instructions present in this document are not followed.

6.3. Complaints, return requests, non-conformities, incidents

In case you want to file a complaint, report a non-conformity or an incident, MotionTech provides you with several channels:

- **Email:** you can send your request at contact@motiontech.ch
- **Your® Liner Follow-up Platform:** your prosthetic technician can file complaints, return requests, report non-conformities or incidents online on the Your® Liner Follow-up Platform: <https://motiontech.ch/yourlinerfollowupplatform>

7. Warnings & Precautions

⚠ Your® Liner should not be worn for more than **8h** in a row. After that time, Your® Liner should be doffed and cleaned as aforementioned.

Your® Liner has been designed specifically for you. It is intended only for your usage, not for other patients. Use of Your® Liner by multiple patients could lead to cross contamination between patients. Other patients may also cause themselves injuries if using your liner, due to unadapted shape.

⚠ Your® Liner is intended to be in contact with and only with the skin of the stump it was designed for. If you have a bilateral amputation, do not try Your® Liner on the limb it was not intended for.

The skin on which Your® Liner is placed needs to be intact. If any wounds, rashes, redness, eruptions, blisters or any other injuries are present on your limb, do not wear the liner until the injuries are healed. Consult your prosthetic technician or physician if you have persistent skin injuries that may require professional medical attention.

To avoid danger of suffocation, keep Your® Liner away from infants and children, and do not try donning the liner on your head in any case.

Your® Liners can melt or burn if exposed to high temperatures or flame. Do not expose your liner to these conditions.

Your® Liners shall be used only for the purpose and in the manner described in this document. MotionTech declines all responsibility if any harm, material or physical, occurs while using its products for another purpose.

Your® Liners are made of silicone materials. If you are allergic to silicone, do not use a Your® Liner.

Your® Liners are tailor made based on specific design prescriptions. Depending on this design prescription, on usage conditions and on your activity level, their lifetime can differ substantially.

After using Your® Liner for 6 months, you should contact your prosthetic technician for replacement, even if it seems to be in good condition. Wearing a deteriorated Your® Liner may put you at risk of rashes and other skin problems.

When disposing of a worn out Your® Liner or providing it to anyone, please thoroughly implement the cleaning and disinfections procedures described in section 4. Once free of potential traces of blood, excretions or secretions, Your® Liner can be disposed of as normal domestic waste, in accordance with your local waste disposal regulations.

Should any serious incident in relation to Your® Liner occur, please report it immediately to MotionTech and to your national competent authority for medical devices.

Your® Pin-lock Liners include metal parts. Do not wear Your® Pin-lock Liner if you engage in a therapeutic procedure which involves magnetic fields and/or radiation.

Your® Liner has been designed specifically for the shape of your limb. If your residual limb has undergone significant volume changes since your liner was delivered to you, you should stop using it. Using an un-fitted Your® Liner may put you at risk of injuries, or of your limb slipping out of the liner.

When you engage in an activity that could result in excessive perspiration make sure to periodically remove your liner to dry it off. Similarly, make sure both your stump and your liner are dry before donning. Wearing a wet liner involved in the suspension of the prosthesis (i.e. Pin-lock and Suction liners) exposes you to the risk of falling.

If using it underwater, water might come between your limb and Your® Liner. The liner risks slipping off your limb and your risk losing your prosthesis.

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